

July 1, 2025

RE: Release of BG Analytics® Software Update Version 1.2.15



Dear Valued Customer:

We're excited to share with you the next release of the Fungitell STAT® BG Analytics® Software!

BG Analytics® is a unique, easy-to-use, intuitive software application that provides a qualitative measurement of (1→3)-β-D-Glucan for use with the Fungitell STAT® kinetic assay. BG Analytics® Software collects and processes data from Lab Kinetics 8-well incubating tube reader, stores the information in a database and produces reports of the test results.

BG Analytics® Version 1.2.15 Release Notes

- Added the Fungitell® predicate pg/mL estimated value associated with the Fungitell STAT® index/category information for reference only in the BG Analytics® Test Result "Note: When using the PKF08 with BG Analytics® Software, the Fungitell STAT® index result is used to determine categorization. An estimated pg/mL value is provided for reference only."
- Added kinetic trace curve results to the report for the Fungitell STAT® Standard (internal control) in case of an Invalid Standard result.
- Updated the BG Analytics® Software to Windows 11.
- Updated the screenshots to the registered trademark symbol.

BG Analytics® is a digitally distributed software application that can be downloaded from the Associates of Cape Cod, Inc., (ACC) software portal on the ACC website at <https://portal.acciusa.com>.

Should you require additional information, please contact me at cjones@acciusa.com or your local representative. A digital copy of this letter along with all other product notifications can be found on our corporate website at www.acciusa.com under the "News & Notices" section.

We appreciate your business and will continue to provide you with industry leading products and services to meet your current and future needs.

Sincerely,

Celeste P. Jones
Clinical Diagnostic Product Manager

MKT# 25-306